Ferno SlideLock - Features

- 1. Provides the flexibility of where trolley can be positioned in a vehicle
- Allows a vehicle to be quickly & easily reconfigured to accommodate a bariatric trolley, such as Harrier LT-LBS or Megasus with the cotsides extended
- Alter lock position 'on-the-fly' to accommodate POWERX with bariatric Split-Scoop surface extender kit attached or with SX cotsides fully deployed horizontal

FERNO









Ferno SlideLock - Operation

To move the trolley lock between the 2 positions:

- 1. Lift the plunger located on the head-end section of the lock (Spike)
- 2. Slide the Spike over to the left (or right), letting go of the plunger once it has started to move
- 3. Continue to move all the way over to the other side and it will lock in position with a positive 'click' sound
- 4. Do the same with the foot-end section, lifting the levers on both sides of the lock and then slide across to the other locking position, letting go of the levers once it has started moving









